COOL COMFORT SLEEVES

COOL COMFORT SLEEVES

COOL COMFORT SLEEVES

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COOL COMFORT SLEEVES

PLEASE READ ALL INSTRUCTIONS BEFORE USING THE PRODUCT

GENERAL APPLICATION TIPS FOR USING THE COOL COMFORT SLEEVES

To ensure the best experience with the Cool Comfort Sleeves, please keep the following in mind:

- **Convenient Storage & Cooling:** Keep the included zip bag—it serves as a practical transport and cooling case, ensuring your sleeve is always ready for use.
- **Dry Use:** Keep the Cool Comfort Sleeve dry to maintain the material's quality over time.
- **Individual Sensitivity to Cold:** Everyone experiences cold differently. Adjust the cooling time to your personal comfort level.
- **Safety First:** If you are unsure how to use the Cool Comfort Sleeve for your specific needs, consult your doctor to ensure optimal usage.

WHY USE HEAT THERAPY?

RELAX & RECOVER: THE BENEFITS OF HEAT APPLICATION

Applying heat is a natural and effective way to relax, recover, and recharge – especially if you experience muscle tension, stiffness, cramps, or everyday aches.

BENEFITS OF HEAT THERAPY:

- Relieves Muscle Tension - Gently relaxes stiff, overworked muscles.
- Improves Circulation - Promotes better blood flow for faster recovery.
- Relieves Cramps and Pain - Helps with joint pain and everyday aches.
- Promotes Mobility - Makes movement easier by loosening tight areas.
- Promotes Relaxation - A calming, cozy way to relieve stress and relax.

Heat therapy can provide you with immediate comfort and long-lasting relief.

AREA OF APPLICATION

- Muscle tension and cramps
- Muscle soreness
- Pain in the muscles and joints

EFFECTS

- Relieves muscle tension
- Improves blood circulation
- Relieves cramps and pain
- Promotes mobility
- Promotes relaxation

CONTRAINDICATIONS

- Lipedema/lymphedema patients: Increased swelling due to dilated blood vessels
- Acute inflammation: Slower healing process
- Fresh injuries: Heat increases blood flow and swelling
- Circulatory disorders (arterial vascular diseases)

INSTRUCTIONS FOR HEAT APPLICATION

MICROWAVE

1. Remove the Venen Engel Cool Comfort Sleeve from its packaging.
2. Place it in the microwave for about 20 seconds.

Important!

- a) Heating time may vary depending on the microwave. Heat in 10-second increments and check the temperature to ensure it is comfortably warm, not hot.
3. If necessary, continue heating in short intervals until the desired temperature is reached.

Important!

- a) Do not microwave for more than 1 minute at a time or 2 minutes in total
- b) If the sleeve is frozen, let it thaw at room temperature before heating.

Value	Time
650–799 W	120 seconds
800–1000 W	90 seconds
Do not heat the Cool Comfort Sleeve in the microwave directly after toting it in the refrigerator/freezer, but only when the Cool Comfort Sleeve itself has reached room temperature.	

WHY USE COLD THERAPY?

COOL COMFORT: THE POWER OF COLD APPLICATION TO RELIEVE LYMPHEDEMA AND LIPEDEMA

If you're dealing with lymphedema or lipedema, you know how uncomfortable swelling, inflammation and tenderness can be. That's where cold comes in – a simple yet powerful way to find natural relief while supporting your body's healing process.

BENEFITS OF COLD APPLICATION

- Minimizes Swelling – Reduces excess fluid buildup.
- Reduces Pain & Sensitivity – Numbs discomfort.
- Calms Inflammation – Helps reduce flare-ups.
- Enhances Circulation & Lymph Flow – Encourages fluid removal.
- Provides Instant Relief – Quick and effective.

Incorporating cold therapy into your routine can make a big difference to your daily well-being and mobility.

AREA OF APPLICATION

- Muscle tension and cramps
- Muscle soreness
- Pain in the muscles and joints

EFFECTS

- Minimizes swelling
- Relieves discomfort
- Reduces inflammatory reactions
- Promotes blood circulation
- Quick relief

CONTRAINDICATIONS

- Fresh injuries
- Open wounds

INSTRUCTIONS FOR COLD APPLICATION

FRIDGE

1. The Cool Comfort Sleeve should not be too cold for cold application. A temperature of 2-8 degrees is optimal.
2. Place your Cool Comfort Sleeve in a resealable bag (e.g. in the bag provided or other sealable containers) in the refrigerator.
3. Cool the Cool Comfort Sleeve for at least two hours. You can simply store the Cool Comfort Sleeve in the refrigerator until you are ready to use it.
4. **Tip:** If you want to cool down on the go straight after your workout, simply take your Cool Comfort Sleeve with you in a small cooling bag.

FREEZER

1. Need it quickly? Then you can also cool your Cool Comfort Sleeve in the freezer.
2. Place your Cool Comfort Sleeve in a resealable bag (e.g. in the bag provided or other sealable containers) in the freezer.
3. Cool the Cool Comfort Sleeve for a maximum of 20 minutes.
4. **Caution:** Make sure that the Cool Comfort Sleeve does not get too cold in the freezer. The cooling should still feel pleasant.

CARE INSTRUCTIONS

Use a damp cloth to gently clean the surface of your Cool Comfort Sleeves. Then allow the Cool Comfort Sleeve to air dry.

The Cool Comfort Sleeve must NOT be put in the washing machine or dryer!

SYMBOLS

Symbol	Definition
	Manufacturer
	Manufacturing country: China
	Reseller
	Importer
	Carefully read the instructions for use
	Do not machine wash
	Do not iron
	Do not bleach
	Handwash only
	Do not tumble dry
	Dispose of product in accordance with local regulations.
	RoHS conformity
	Disposal instructions France

PRODUCT INFORMATION

Product dimensions	LxB: 79,4 x 61,3 cm	
Weight	3000 g	
Thigh circumference recommendation	70 cm (+- 5 cm)	
Material	85 % Nylon 15 % Spandex	
Fit (+5 cm buffer)	Ankle 21 cm Calf 40 cm	Knee 53 cm Thigh 68 cm

WARRANTY STATEMENT

We provide a 24-month guarantee on the product. Unfortunately, no guarantee claims can be made for defects that are the result of improper shipping. Guarantee conditions: If there is a legal obligation to provide a guarantee, the right to a replacement or a reduction in price is excluded, and the manufacturer will either be given the opportunity to make improvements free of charge. If the improvement is unsatisfactory despite several attempts, or if a replacement product has a defect for which the manufacturer is responsible, the customer has the right to a replacement or a reduction in price.

Warranty exclusion: The warranty does not cover defects caused by violent damage, improper operation, external force or modifications and repairs carried out by third parties, such as defects caused by incorrectly dimensioned or short-circuited fuses or defects due to normal wear and tear.

Warranty requirement: The warranty claim only exists in conjunction with the original purchase receipt (invoice). Therefore, keep the purchase receipt carefully. If you have any problems or questions when using the system, please do not hesitate to contact us at +49 (0) 666 174 891 05 or service@venenengel.com.

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Wellcosan GmbH