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OPERATING INSTRUCTIONS

## VENEN ENGEL 6 MOBIL

DE / EN / FR / ES / IT

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ENGLISH

## PRELIMINARY REMARK

**Congratulations on the purchase of this device!** You will enjoy your Venen Engel very much! If you are satisfied with your Venen Engel, we would be very pleased if you submitted a review.

This guide includes details that should be taken into account by the user, to avoid risks, and to enable the safe use of the device.

Should you have any questions about its application, its accessories, or if you have suggestions, please do not hesitate to contact us! We are also happy to help you with any complications that develop during or after using the product.

**Telephone:** +49 (0) 666 17 48 91 05

**Whatsapp:** +49 1573 599 0084

**Mail:** [service@venenengel.de](mailto:service@venenengel.de)

**Web:** [www.venenengel.de](http://www.venenengel.de)

**HelpDesk:** [help.venenengel.de](http://help.venenengel.de)

## BEFORE YOU GET STARTED

- Please read the operating instructions carefully before using the device.
- Keep this user manual so you can consult it, if necessary.
- Venen Engel may not be used if you suffer from certain health conditions. Please read the list of contraindications carefully on last page.

## STEP 1: BEFORE USE

### CONNECT THE HOSE SET AND CUFFS

Connect the tubes to the connectors on the cuffs. Make sure the longest tube goes to the foot air chamber.

**Note:** We recommend that you do not detach the connectors after setup.

### CONNECT THE TUBE SET AND THE CONTROL UNIT

There are two clips on the side of the multi-plug. Depress the clips then insert the multi-plug into the control unit.

**Note:** Please insert the small fittings of the multi-plug into the openings found above the air outlets.


### CONNECT THE POWER SUPPLY TO THE CONTROL UNIT

Connect the power supply to both the control unit and an outlet to ensure power.

**Note:** To keep the device clean, we recommend wearing thin jogging pants or leggings. Keep pockets empty. Zip the cuffs completely closed to avoid sudden opening. Be careful not to kink the tubing during use.


## STEP 2: OPERATION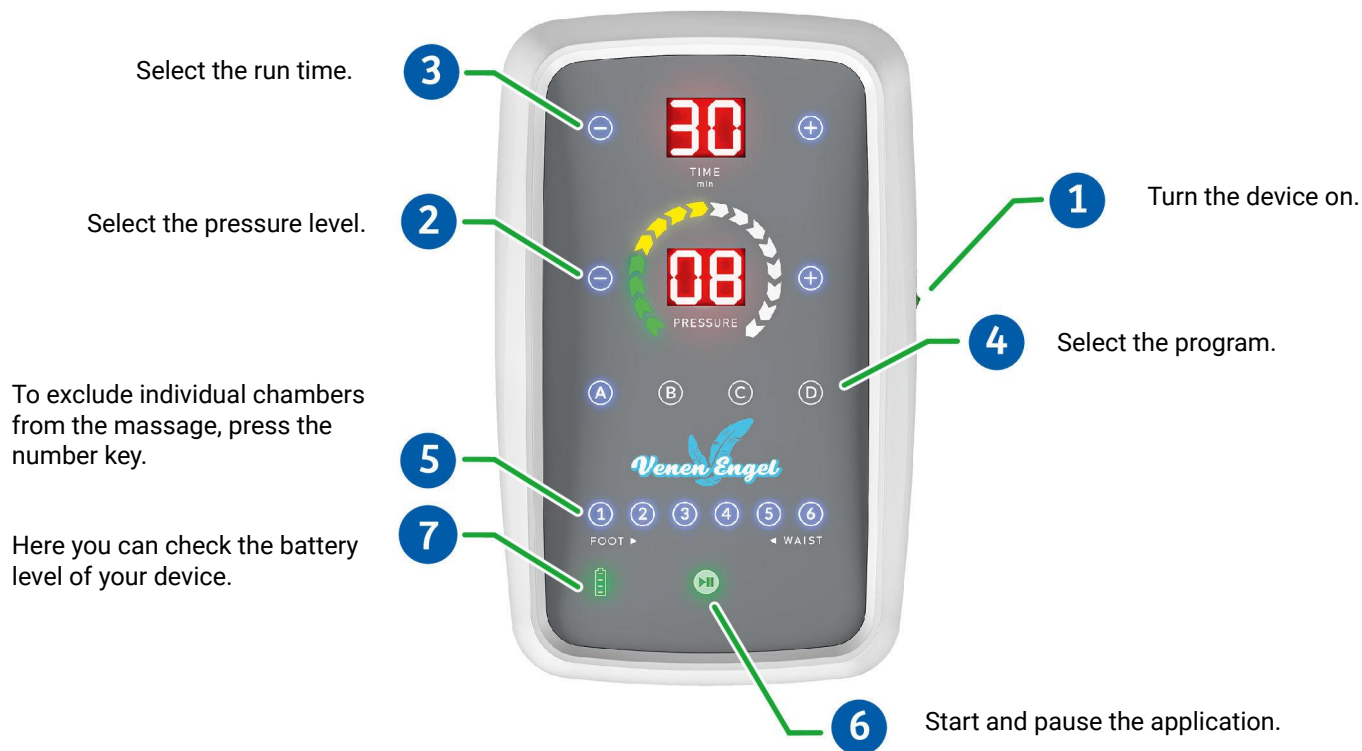

## STEP 3: ENJOY THE MESSAGE

### MESSAGE PROGRAMS A-D:

The air chambers in the leg cuffs are filled based on the selected program. This creates an optimal and relaxing gliding pressure message on both legs. Duration and pressure, in addition to optional additional functions can also be adjusted during the message.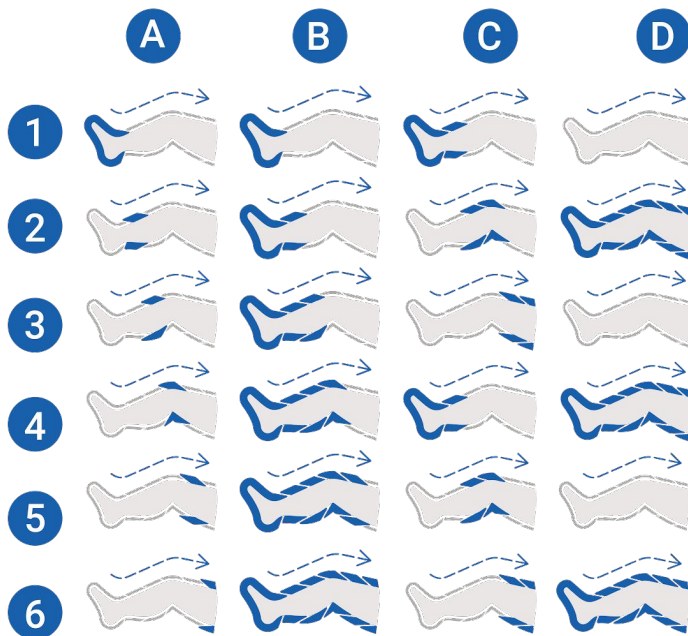

## STEP 4: STOPPING AND DISASSEMBLY

After the set time has elapsed or after pressing the off switch, remove the multi-plug from the device. If you don't feel well during the message, you can switch the system off at any time.

Once turned off, you can open the cuffs. To quickly deflate the air chambers after use, press on the air chambers with both hands. The gray tube cuff connectors can be left permanently connected. For storage, place the cuffs (incl. tubes) on top of each other. Then roll them loosely together. The cuffs can be cleaned using a damp cloth and mild detergent. Allow the cuffs to dry thoroughly. Disinfectants such as Descosept pur or comparable products with a similar composition are suitable for disinfecting the cuffs.

## TROUBLESHOOTING

### AIR IS ESCAPING (AUDIBLY)

1. Check the tubes and connectors for damage or kinks that could cause air to be trapped.
2. Make sure the multi-plug is seated correctly in the control unit.

### THE CUFFS INFLATE IN THE WRONG ORDER

1. Make sure the multi-plug is inserted in the control unit using the correct orientation. The multi-plug is seated correctly if the small fittings of the multi-plug are in the openings found above the air outlets.

## **I HAVE A LEG / ABDOMEN SET AND WISH TO USE ONLY THE LEG OR ABDOMEN CUFFS THIS TIME**

If you would like to use your Vein Angel in a different configuration than the one that was shipped to you, you will need the appropriate hose set. All hose sets are available in our online shop. If you need help selecting a set, please contact us! We will be happy to help you.

## **THE DEVICE PUMPS AIR, BUT NEITHER OR ONLY ONE OF THE CUFFS IS INFLATING**

If, after starting the programme, you find that not all the sections are filling with air, please check (1) if the multi-plug is correctly inserted into the controller. Also check that (2) the air hoses are not kinked and that you are not sitting on any of them. Finally, check (3) that the plugs in the cuffs are all properly inserted.


# Instruction for Accessories

## ATTACHING THE EXTENSION FLOP


## PUTTING ON THE ARM CUFF

Step 1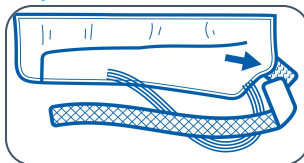

Step 2

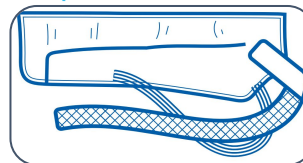
**Velcro the strap to the back of the cuff.**

Step 3

Step 4

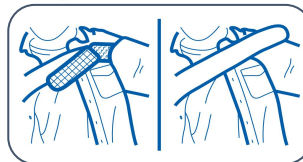**Pull the cuff onto your arm and make sure the end is sitting on your shoulder. Run the strap under your other arm and attach it to the front of the cuff.**

## PLACEMENT OF THE ABDOMEN CUFF

The bottom of the cuff should sit directly against the top of the leg cuffs. **Tip:** put on the leg cuffs first. Then place the abdomen cuff behind you on the ground (inside facing up). Lay yourself back onto the cuff. Once you have adjusted it to the correct height, zip it around your stomach.

## COMBINING PIECES

If you would like to use additional cuffs, make sure you pay attention to the number of air chambers in each one as you buy them. If you would like to use the additional cuffs in combination with the ones you have, you will need the appropriate hose set.

## CONTENT & TECHNICAL DETAILS

Device

Power plug

Cuffs

Multi-Plug air hose set

Number of programmes	4
Number of air chambers	6
Weight [in kg]	-1,6
Power output [in W]	35
Mains frequency [in Hz]	50 / 60
Voltage [in V]	AC110-240
Timer setting [in min]	5-99
Pump output [in l/min]	12
Pressure [in mmHg]	30-240
Dimensions [in cm]	22 x 14 x 9
Protection class	II/ 
Temperature range [in °C] (use and storage)	10-40
Humidity [in %] (use and storage)	30-85
Battery: <ul style="list-style-type: none"> <li>• Charging time / running time [in min]</li> <li>• Type</li> <li>• Model number</li> <li>• Voltage</li> <li>• Capacity</li> <li>• Stored current</li> </ul>	130 / 180 Li-ion 18650 11.1 V 5000 mAh 55.5 Wh

## SAFETY INSTRUCTIONS

Keep these operating instructions in a safe place! Do not disassemble or burn the unit. Repairs to the control unit may only be carried out by specialists authorised by Wellcosan GmbH, otherwise the warranty may be voided. In case of damage to the unit, pull out the mains plug and switch off the unit. Also disconnect the mains plug and switch it off if the unit has come into contact with water or other liquids. Do not use the unit in damp rooms. Ensure that the unit is standing securely. Do not subject the device to shocks or throw them. The unit must not be covered by other objects during operation. Make sure that the hoses do not twist or kink during use and storage. Keep the unit away from sources of heat (e.g. direct sunlight, radiators and others). When you are not using the unit, the unit must be switched off. The unit switches itself off after 10 minutes of inactivity. When not in use also unplug the appliance from the mains. Always unplug the appliance from the mains before cleaning. Keep the product away from children or persons who have not been instructed in its use.

With the CE marking, the manufacturer or importer confirms the conformity of the product with the applicable EC directives and compliance with the "essential requirements" specified therein.

This marking on the unit means that you must not dispose of it in normal household waste at the end of its service life. Please have it disposed of by an electronic waste disposal company.

The abbreviation UKCA stands for UK Conformity Assessed. The marking is a prerequisite for the marketing of products within Great Britain.

## WARRANTY

We grant 24 months on the control unit. In the event of a malfunction, it may be necessary to send the unit in for inspection. In this case, please ensure that the shipping carton is appropriately padded in order to avoid possible transport damage. Unfortunately, no warranty claims can be made for defects that are due to improper shipping. **Warranty conditions:** Insofar as there is a legal obligation to provide a warranty, either a replacement will be provided free of charge or the manufacturer will be given the opportunity to rectify the defect, to the exclusion of the right to rescission or reduction. Should the rectification of defects fail despite several attempts or should a replacement unit also be afflicted with a defect for which the manufacturer is responsible, the customer shall be entitled to rescission or reduction.

**Warranty exclusion:** Excluded from the warranty are defects caused by forcible damage, improper operation, external force or third party modification and repair measures, such as defects caused by incorrectly dimensioned or short-circuited fuses or defects attributable to normal wear and tear.

**Warranty requirement:** The warranty claim exists only in conjunction with the original proof of purchase (invoice). Therefore, keep the purchase receipt in a safe place.

If you have any problems or questions when using the system, please do not hesitate to contact us at +49 (0) 666 17 48 91 05 or [service@venenengel.de](mailto:service@venenengel.de).

## LIABILITY DISCLAIMER

The product should ideally only be used after consultation with your doctor.

**Possible contraindications may include:** Decompensated heart failure, cardiac output disorders with water in the legs, inflammation of the treated areas of the body, known or suspected leg vein thrombosis, phlebitis and arterial disease, gangrene, dermatitis, arteriosclerosis or venous circulatory disorders, open, untreated or infected wounds and injuries, for leg pain, skin lesions, acute inflammation, malignant tumours, pulmonary oedema, hypertension, extensive thrombophlebitis, thrombosis or suspected thrombosis, erysipelas, severe uncontrolled hypertension, acute soft tissue trauma of the extremities, neuropathy or occlusive processes in the lymphatic drainage system.

**Purpose:** The product is designed to help you feel better and relax.

### NOTICE:

Venen Engel 6 Premium is not a medical device and is not intended for self-treatment. If you are unsure, please consult your doctor.

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© Wellcosan GmbH 2022
Wellcosan GmbH, Hauptstraße 27