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Thank you for choosing a product from Venen Engel!

Discover the **Venen Engel compression stockings**, which bridge the gap between health and a modern lifestyle. Thanks to the high-quality material, they are comfortable to wear and can be easily integrated into everyday life without compromising on comfort. They support blood circulation and are a stylish addition to any outfit. Choose between **toeless knee-high socks** (underknee) or **toeless thigh-high socks** (stay-up) as required.

Each pair of Venen Engel compression stockings is designed to be not only functional but **also visually appealing, with a choice of colors ranging from classic to modern.**

Venen Engel compression stockings are classified as a medical device and should be worn in accordance with the instructions for use. For optimum effect and safety, we recommend that you familiarize yourself with the specific properties and instructions for use.

The Venen Engel **class II compression** stockings have a **pressure of 23-32 mmHg**, whose dynamic effect of graduated compression ensures a light and comfortable feel. During wear, the circulation is gently stimulated in order to facilitate venous return. The compression decreases from the ankle upwards.

The Venen Engel compression stockings have a reinforced heel, which offers you the following advantages:

- The proper placement of the heel in the heel pocket, thus providing for greater stability.
- Proper positioning at the maximum compression point, as well as a better fit, greater wearability and elasticity around the circumference of the ankle.

Directions

- Venen Engel compression stockings are recommended as **useful aids for preventing venous disorders.**
- They are particularly effective for **relieving the sensations of heaviness and fatigue in the legs**, as well as for **reducing serotonin-induced oedema.**

- With consistent use, they can even **help prevent or delay the occurrence of venous valvular insufficiency** in susceptible individuals.
- **They are suitable for pregnant women and for some forms of initial distal insufficiency of the superficial veins** in young or middle-aged people who have to stand a lot or are genetically predisposed.

Please discuss with your doctor in advance whether you have any contraindications to the use of medical compression stockings.

General contraindications

- Advanced state peripheral arterial occlusive disease
- Massive oedema of the lower limbs of a pathological nature
- Septic phlebitis
- Severe phlebitis, such as phlegmasia cerulea dolens
- Lymphangitis
- Advanced peripheral neuropathies (i.e., diabetes mellitus)
- Other sensory disorders of the limbs
- Intolerances/allergies related to the material of the compressive stockings.

Relative contraindications

- Ulcers and open wounds
- Moist dermatosis
- Immobility
- Dermatitis stasis
- Valvular incontinence of the superficial and deep venous system

- Deep vein thrombosis or acute thrombophlebitis
- Post thrombophlebitic syndrome
- For stockings with silicone edges, skin irritation may arise in cases of hypersensitivity of the material itself

Special warnings

Medical compression stockings should only be worn after consulting a doctor. It is advisable to seek expert advice before purchasing. In the event that pain or skin irritation should occur when wearing the compression stockings, remove the stockings immediately and consult a physician or your retailer.

Instructions and recommendations for wearing Venen Engel compression stockings

It is recommended to put the stockings on as soon as you get up in the morning.

The stockings should be protected against contact with any oils, creams or ointments applied to the legs for medical or cosmetic reasons. Rough fingernails or rings with rough edges could potentially damage the product's fabric. If the skin is wet, apply talcum powder to the foot in order to facilitate its insertion into the stocking.

- 1.** Turn the stocking inside out, with the exception of the foot section. It is best to turn the stocking inside out to the point that the heel remains outside.
 - Use both hands to widen the foot portion, and slide the foot into the stocking up to the heel.
 - Then roll the stocking upwards along the leg, positioning the heel correctly.
 - Turn back the inside-out portion of the stocking, taking care to properly position the heel

2. Insert the rest of the leg up to the knee, smoothing out the fabric with the palm of your hand. Never pull directly on the top edge of the stocking.

- In the case of tights, follow the same procedure as described above until the tights arrive at the level of the knee on both legs.

3. Roll the remaining portion of the tights upward to the crotch, smoothing the fabric with your hands and checking to make sure the compression is uniform.

Tip: To put on the stockings with toe opening, use the slip stocking applicator included in the scope of delivery.

Composition of the materials

The textile composition of each individual product is indicated on the outside of the package. The materials with which our products are manufactured are listed hereafter.

CO: Cotton - **EA:** Elastane

PA: Polyamide - **WO:** Wool - **PP:** Polypropylene

Product duration

With regular use, the shelf life of medical compression stockings is up to six months. Do not use the product beyond the specified expiry date, indicated on the packaging.

Washing instructions

Venen Engel stockings can be washed by hand or in the washing machine.

- **Hand washing:** use mild detergents for delicate fabrics. Rinse thoroughly. Do not wring. Air dry.
- **Washing machine:** place the garment in an appropriate bag or in a pillow case. Do not bleach. Do not machine dry

Do not dry clean. Do not iron.

Note

The manufacturer shall bear no responsibility for any personal injuries and/or property damage that may occur in the event that the product is used improperly or for purposes other than those for which it is intended.

If you have any questions or uncertainties, please contact your doctor or specialist dealer.

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Sanyleg Srl a Socio Unico
Wellcosan GmbH